test traka

SRB

Test za rano utvrđivanje trudnoće – za samotestiranje

Princip

Babyrapid® hCG test za rano utvrđivanje trudnoće je brzi, imunotest, bočnog protoka, u obliku test trake, za kvalitativno otkrivanje humanog horionskog gonadotropina (hCG) u urinu u cilju utvrđivanja trudnoće.

Test koristi kombinaciju antitela, uključujući monoklonalna hCG antitela, kako bi se selektivno otkrili povišeni nivoi hCG.

Test se vrši uranjem test trake u urin, a rezultati se očitavaju u delu kontrolne i test linije.

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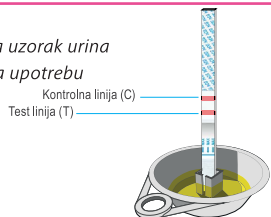
Molimo pročitajte sve informacije koje se nalaze u ovom pakovanju pre nego što uradite test.

- Ne koristiti nakon datuma isteka koji je odštampan na kesici.
- Čuvati na suvom mestu na temperaturi od 2-30°C ili 36-86°F. Ne zamrzavati.
- Ne koristiti ako je kesica pocepana ili oštećena.
- Čuvati van domašaja dece.
- Samo za in vitro dijagnostiku. Ne koristiti interno.
- Ne otvarati kesicu sa test trakom pre nego što ste spremni da započnete sa testom.

Upotrebljeni materijal za testiranje odstranite u skladu sa lokalnim propisima.

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- Test traka
- Posudica za uzorak urina
- Uputstvo za upotrebu

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Pozovite: +49-4102-8009-0

Reference

1. Batzer FR. Hormonal evaluation of early pregnancy, *Fertil. Steril.* 1980; 34(1): 1-13
2. Catt KJ, ML Dufau, JL Vaitukaitis Appearance of hCG in pregnancy plasma following the initiation of implantation of the blastocyte, *J. Clin. Endocrinol. Metab.* 1975; 40(3): 537-540
3. Braunstein GD, J Rasor, H. Danzer, D Adler, ME Wade Serum human chorionic gonadotropin levels through-out normal pregnancy, *Am. J. Obstet. Gynecol.* 1976; 126(6): 678-681
4. Lenton EA, LM Neal, R Sulaiman Plasma concentration of human chorionic gonadotropin from the time of implantation until the second week of pregnancy, *Fertil. Steril.* 1982; 37(6): 773-778
5. Steier JA, P Bergsjo, OL Myking Human chorionic gonadotropin in maternal plasma after induced abortion, spontaneous abortion and removed ectopic pregnancy, *Obstet. Gynecol.* 1984; 64(3): 391-394
6. Dawood MY, BB Saxena, R Landesman Human chorionic gonadotropin and its subunits in hydatidiform mole and choriocarcinoma, *Obstet. Gynecol.* 1977; 50(2): 172-181
7. Braunstein GD, JL Vaitukaitis, PP Carbone, GT Ross "Ectopic production of human chorionic gonadotropin by neoplasms", *Ann. Intern Med.* 1973; 78(1): 39-45

ulti med Products (Deutschland) GmbH, Reeshoop 1, 22926 Ahrensburg, Germany www.ultimed.org

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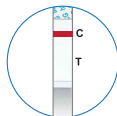
Pozovite: +49-4102-8009-0

Reference

1. Batzer FR. Hormonal evaluation of early pregnancy, *Fertil. Steril.* 1980; 34(1): 1-13
2. Catt KJ, ML Dufau, JL Vaitukaitis Appearance of hCG in pregnancy plasma following the initiation of implantation of the blastocyste, *J. Clin. Endocrinol. Metab.* 1975; 40(3): 537-540
3. Braunstein GD, J Rasor, H. Danzer, D Adler, ME Wade Serum human chorionic gonadotropin levels throughout normal pregnancy, *Am. J. Obstet. Gynecol.* 1976; 126(6): 678-681
4. Lenton LA, LM Neal, R Sulaiman Plasma concentration of human chorionic gonadotropin from the time of implantation until the second week of pregnancy, *Fertil. Steril.* 1982; 37(6): 773-778
5. Steier JA, P Bergsjö, OL Myking Human chorionic gonadotropin in maternal plasma after induced abortion, spontaneous abortion and removed ectopic pregnancy, *Obstet. Gynecol.* 1984; 64(3): 391-394
6. Dawood MY, BB Saxena, R Landesman Human chorionic gonadotropin and its subunits in hydatidiform mole and choriocarcinoma, *Obstet. Gynecol.* 1977; 50(2): 172-181
7. Braunstein GD, JL Vaitukaitis, PP Carbone, GT Ross "Ectopic production of human chorionic gonadotropin by neoplasms", *Ann. Intern Med.* 1973; 78(1): 39-45

ulti med Products (Deutschland) GmbH, Reeshoop 1, 22926 Ahrensburg, Germany www.ultimed.org

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Za dodatna pitanja obratite se proizvođaču uliti med Products (Deutschland) GmbH koji će Vam rado pomoći.